



1

Lay the 'Optivizor' flat with label and buckle up.

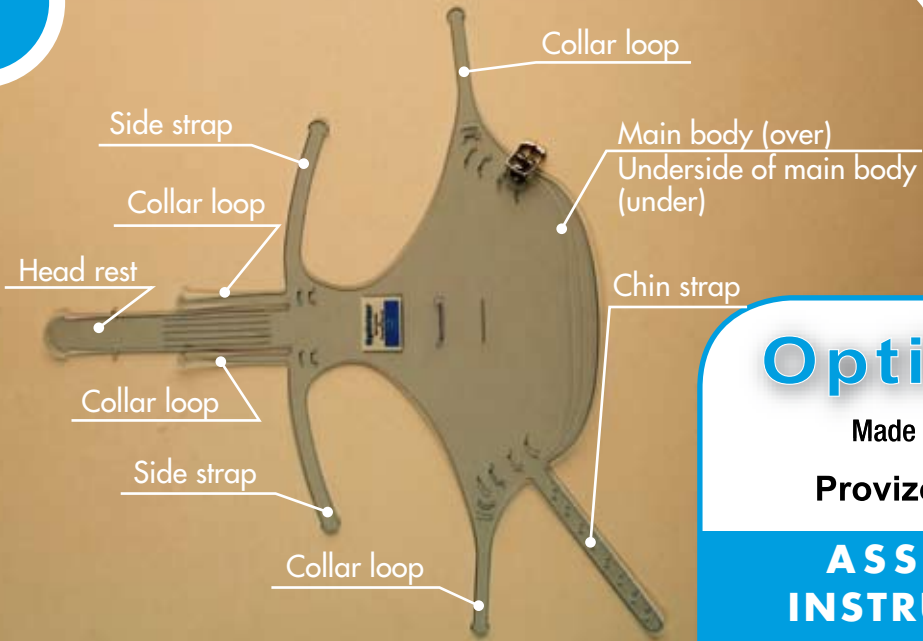

2

Insert head rest under main body through slot 1A and then through locking slot 1B.

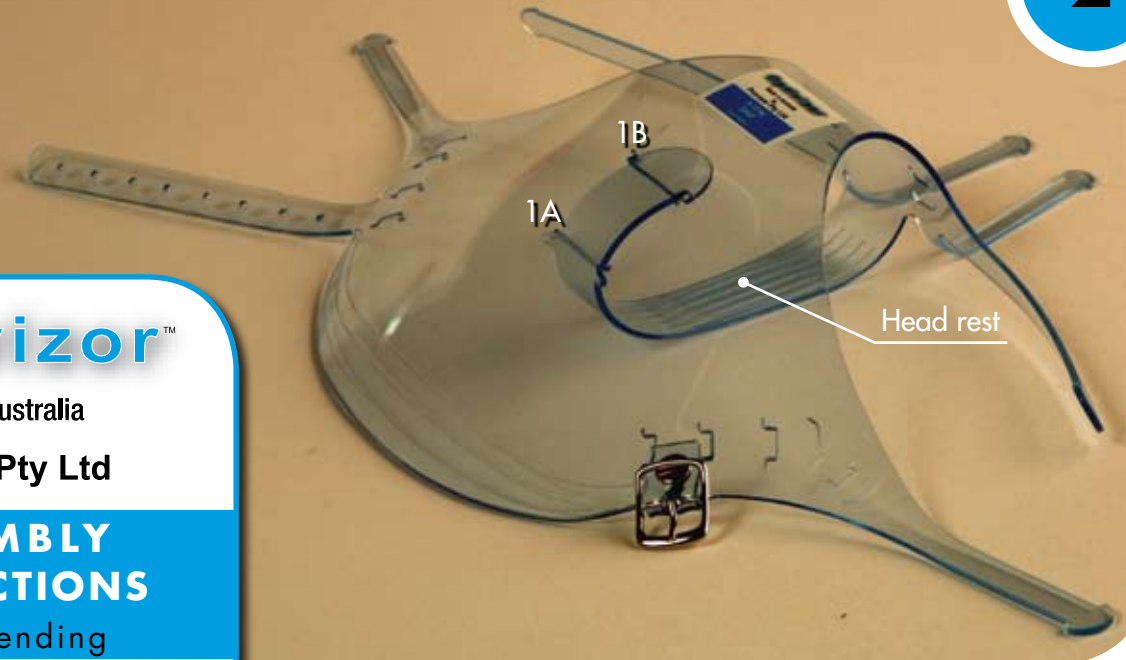

Optivizor™

Made in Australia
by
Provizor Pty Ltd

**ASSEMBLY
INSTRUCTIONS**


Patent Pending




Loosely attach chin strap to buckle, slide 'Optivizor' over the animals head, pull ears through the formed openings and then adjust chin strap to suit.

3

Insert the 2 side straps over the main body of the 'Optivizor' through slots 2A, and then through locking slots 2B, 2C or 2D to suit the animal. Insert the 4 collar loops over the main body through slots 3A, and then through locking slots 3B.

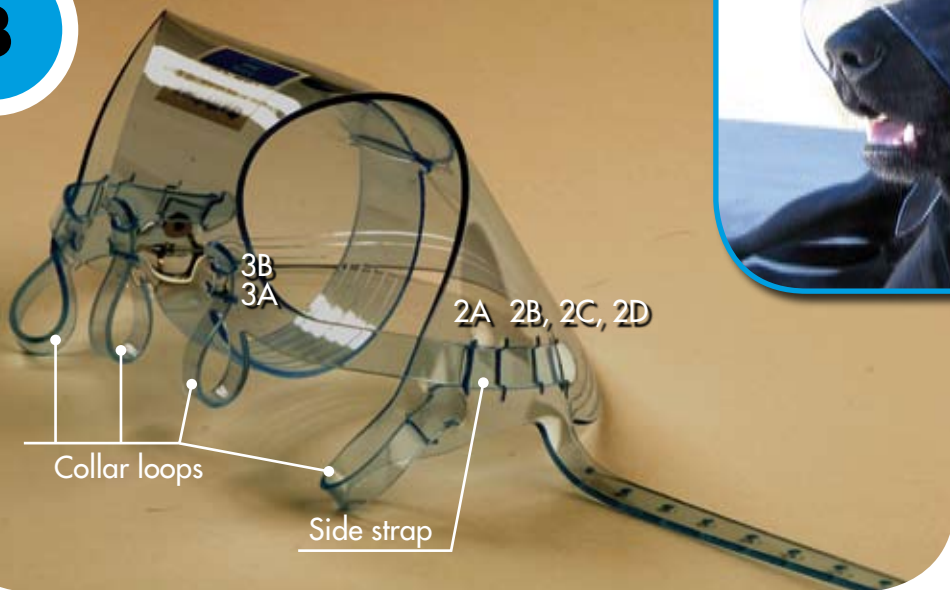

4

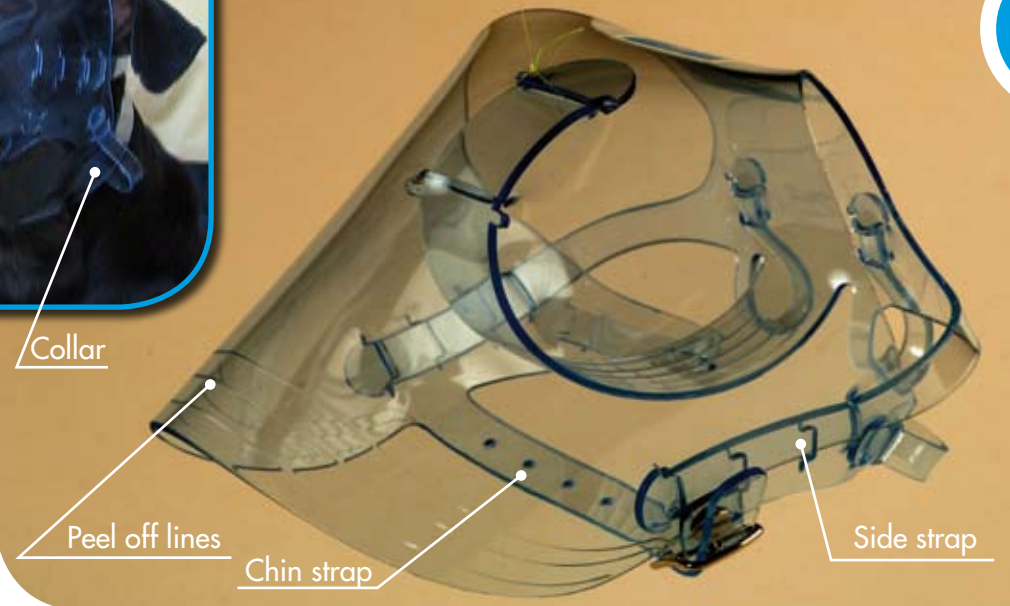

5

Refer central photo. Thread collar (supplied with 'Optivizor') through collar loops shown in diagram 3 to secure the 'Optivizor' to the animals head and adjust side and chin straps to suit.

6

Snip both ends off at the selected peel off line if necessary and peel off as indicated on diagram 4 to adjust for length of snout.

7

Cleaning. Simply immerse in warm soapy water. Wipe, dry and refit.